



Installation Instructions for #3300 30" Van Mount

This kit includes 1) Flat mounting plate 2) Modified first load bearing ramp link with plastic stoppers.

1. Position the flat mounting plate on the floor level of the rear or side door entrance of your van. Typically this should be centered in the opening but you can move it off center if that is needed to provide you with more entry room.
2. Remove the angled approach plate from the upper end of your Roll-A-Ramp and store for possible future use. Save the bolts for the next step. **You will need to remove this approach plate in order to attach the ramp to the bracket assembly.**
3. Attach the first upper load bearing link of the ramp to the mounting bracket. Use the special ramp bolts that secured the upper angled approach plate which was removed in Step 2. Position the bracket and ramp so the ramp does touch either side of the door opening when the ramp is extended.
4. Drill mounting holes through the van floor as provided by the openings in the mounting plate. **Be sure to check for obstructions such as fuel lines, electrical connections before drilling.**
5. The white plastic stoppers on the bracket assembly allow for four (4) separate positions or angles that the ramp is in when in the folded up position. To change the angle, loosen the nut and turn the stopper to desired angle, then re-tighten the nut.

Convert Roll-A-Ramp to a bi-fold ramp

1. To determine the pivot point for your ramp put the Roll-A-Ramp in an upright position using one link as the pivot point and the lower angled approach plate resting on the floor.
2. Mark the pivot link and extend the ramp to ground level.
3. Tighten the bolts on all links except bolts on either side of the marked pivot link using the Allen wrench and 1/2" wrench. (These are provided with the Roll-A-Ramp owner's manual.) Make sure the bolts on either side of the link where you want the pivot point to be are not tightened. This will allow the ramp to be folded in a bi-fold manner.

With these links tightened, you will be able to lift the ramp at a mid point and swivel it on the mounted van plate into a position that allows it to be stored upright during transportation. It is recommended that the user fold the ramp by using the slots in the side safety rails.



Pins allow for easy removal of ramp