

7-Degree = Rise / 1.5		10-Degree = Rise / 2		12-Degree = Rise / 2.4		15-Degree = Rise / 3	
RISE Distance (inches)	RAMP LENGTH	RISE Distance (inches)	RAMP LENGTH	RISE Distance (inches)	RAMP LENGTH	RISE Distance (inches)	RAMP LENGTH
7"	5 FT	7"	4 FT	7"	4 FT	7"	3 FT
8"	6 FT	8"	5 FT	8"	4 FT	8"	3 FT
9"	7 FT	9"	5 FT	9"	5 FT	9"	4 FT
10"	7 FT	10"	6 FT	10"	5 FT	10"	4 FT
11"	8 FT	11"	6 FT	11"	5 FT	11"	4 FT
12"	9 FT	12"	7 FT	12"	6 FT	12"	5 FT
13"	9 FT	13"	7 FT	13"	6 FT	13"	5 FT
14"	10 FT	14"	8 FT	14"	7 FT	14"	5 FT
15"	11 FT	15"	8 FT	15"	7 FT	15"	6 FT
16"	11 FT	16"	9 FT	16"	7 FT	16"	6 FT
17"	12 FT	17"	9 FT	17"	8 FT	17"	6 FT
18"	13 FT	18"	10 FT	18"	8 FT	18"	7 FT
19"	13 FT	19"	10 FT	19"	9 FT	19"	7 FT
20"	14 FT	20"	11 FT	20"	9 FT	20"	7 FT
21"	15 FT	21"	11 FT	21"	10 FT	21"	8 FT
22"	15 FT	22"	12 FT	22"	10 FT	22"	8 FT
23"	16 FT	23"	12 FT	23"	10 FT	23"	8 FT
24"	17 FT	24"	13 FT	24"	11 FT	24"	9 FT
25"	17 FT	25"	13 FT	25"	11 FT	25"	9 FT
26"	18 FT	26"	14 FT	26"	12 FT	26"	9 FT
27"	19 FT	27"	14 FT	27"	12 FT	27"	10 FT
28"	19 FT	28"	15 FT	28"	12 FT	28"	10 FT
29"	20 FT	29"	15 FT	29"	13 FT	29"	10 FT
30"	21 FT	30"	16 FT	30"	13 FT	30"	11 FT
31"	21 FT	31"	16 FT	31"	14 FT	31"	11 FT
32"	22 FT	32"	17 FT	32"	14 FT	32"	11 FT
33"	23 FT	33"	17 FT	33"	15 FT	33"	12 FT
34"	23 FT	34"	18 FT	34"	15 FT	34"	12 FT
35"	24 FT	35"	18 FT	35"	16 FT	35"	12 FT
36"	25 FT	36"	19 FT	36"	16 FT	36"	13 FT
Manual wheelchairs, assisted & unassisted walking		Manual wheelchairs, assisted & power wheelchairs		Max recommended for power wheelchairs, or strong assistance w/ manual wheelchairs		ASSISTED ONLY OR RESTRICTED SPACE	
Standard placement = top 12" of ramp is on upper surface. Load-Bearing approach plate = top 6" of ramp is on upper surface.							